

Search for:

Patient Name	Patient ID	Accession	Study ID	Today	Yesterday	Recent	Date Range
Last name: test	First name:						
No filter set							
No filter set							
Full Name	Patient ID	DOB	Gender				
Test, 2	1234567	18-Nov-1858	Female				

1. Find the patient

Progr...	Accession	Performed on	Dictated on	Mod...	Procedure types	Imag...	Reported By	Performed by	Work Group
intra-op	05-Jun-2014 07:44		CR	ABD COMP W/DECUB AND OR...	3				
	10-Nov-2011 10:36		US		19				

2. Right click the study

- Open study ...
- Assign study(s) to ...
- Edit study ...
- Merge studies ...
- Move study ...
- Send study ...
- Prefetch study ...
- Print Study ...
- Choose columns ...

3. Select

- Send study ...
- Send Flagged Images ...
- Manage Send Jobs ...

4. Select

Warning: 300 patients display limit reached. You may need to refine your search. 2 studies found for Test, 2

### Send Tool Options

Send:  
Test, 2 ID:   
Study Id:

Destination:

- Send to AMIPACS
- Life Image (Jefferson)
- ONFPACS
- Powershare (AMI)
- Powershare (MAKO)
- Powershare (Nemours Al DuPont)
- Powershare (NEMOURS FETAL ECHO)
- RadSpa Backup
- RadSpa (Bridgeton)
- RadSpa (Elmer)
- RadSpa (Mullica Hill)
- RadSpa (Vineland)
- RadSpa (Woodbury)
- Send to Ambra (Covera Health)
- Send to Ambra (Penn 'Academy)
- Send to AMIPACS
- Send to Conformis

DO NOT USE THIS!!!

5. USE THIS!!!

1. Find the patient
2. Right click the study
3. Select "Send study"
4. Select "Send study"
5. Select the destination "Send to AMIPACS"
6. Select Send study or studies

### Send Tool Options

Send:  
Test, 2 ID:   
Study Id:

Destination:  
Send to AMIPACS

Send Options

☐ Include All Diagrams

6. Select

Send 1 Study

Cancel